

UltraCheck®

Cleaning and Disinfecting for UltraCheck Reusable Blood Pressure Cuffs

If the blood pressure cuff has been grossly contaminated with blood or other bodily fluids, it should be discarded.

Cleaning Instructions:

For normal cleaning with mild detergents / dilute bleach solution (1-2%), wipe the cuff with the cleaning solution, rinse with water and dry.

Note: If the cuff is submerged in the cleaning solution, the fittings should be capped to prevent the liquid from getting inside the cuff.

Disinfecting Instructions:

Using a disinfecting solution, such as Enzol, Cidezyne, Cidex, Sporidicin, Isopropyl alcohol (70%), Ethanol (70%), wipe or spray the BP cuff and allow to stand approximately one minute, rinse well with water and dry.

Note: If the cuff is submerged in the disinfecting solution, the fittings should be capped to prevent the liquid from getting inside the cuff.

Product Name	Product Number	Product Description
UltraCheck®	REF US/UD0814	Infant
UltraCheck®	REF US/UD1320	Child
UltraCheck®	REF US/UD1826	Small Adult
UltraCheck®	REF US/UD2635	Adult
UltraCheck®	REF US/UD2938	Adult Long
UltraCheck®	REF US/UD3242	Large Adult
UltraCheck®	REF US/UD3544	Large Adult Long
UltraCheck®	REF US/UD4250	Adult Thigh
UltraCheck®	REF US/UD0814MB	Infant Manual Bulb
UltraCheck®	REF US/UD1320MB	Child Manual Bulb
UltraCheck®	REF US/UD1826MB	Small Adult Manual Bulb
UltraCheck®	REF US/UD2635MB	Adult Manual Bulb
UltraCheck®	REF US/UD2938MB	Adult Long Manual Bulb
UltraCheck®	REF US/UD3242MB	Large Adult Manual Bulb
UltraCheck®	REF US/UD3544MB	Large Adult Long Manual Bulb
UltraCheck®	REF US/UD4250MB	Adult Thigh Manual Bulb



Corporate Office:
CAS Medical Systems, Inc.
44 East Industrial Road
Branford, Connecticut 06405 USA
(203) 488-6056

Authorized Representative in EEA:
MediMark® Europe Sarl. BP 2332
38033 GRENOBLE CEDEX 2 - FRANCE

Jacksonville Plant:
Statcorp, Inc.
14476 Duval Place West Suite 303
Jacksonville, Florida 32218 USA
(203) 488-6056

Rx Only



Statcorp

