



Service Bulletin

Date: March 9, 2007

Ref: Daylight Saving Time

Bulletin Number: SB# 07-05

Applicable Model(s): CAS 740 and 750 Monitors

Page: 1 of 3

REASON

Starting in 2007, the United States and Canada will start Daylight Saving Time (DST) on the second Sunday of March (3/11) and end on the first Sunday of November (11/4).

The current software in the monitor will not recognize these new dates if the DST Option in the Monitor's Configuration menu was set to 740 (DST N AM) or 750 (N America). The monitor's Daylight Saving Time setting will need to be set to DST OFF, and the clock adjusted manually to the correct time. The default DST setting used by both monitors is DST OFF.

The procedure to set the Daylight Saving Time option and Clock is provided on the following pages.

SIGN-OFF

TECHNICAL SUPPORT: Anthony Caf
SERVICE MANAGER: Gregory 16720




DATE: 3/9/07
DATE: 3/9/07

Date: March 9, 2007

Bulletin Number: SB# 07-05

Page: 2 of 3

PROCEDURE (740)

To enter the monitor's Configuration menu, press and hold the ALARM LIMITS  and VOLUME  pushbutton keys while the monitor is being turned "ON" . The monitor's DSP software version will be displayed.

NOTE:

While in the Configuration menu, if no pushbutton is pressed within 60 seconds, the monitor will automatically save all changes made and exit the Monitor Configuration menu. The Message Window will briefly display "Saving" and return to the "Ready" mode.

Press the VOLUME pushbutton key until the Time (TM XX:XX) is displayed. The Hour parameter will be flashing.

Press either the UP or DOWN arrow pushbutton keys to change the Hour.

Press the VOLUME pushbutton key until the DST menu option is displayed.

Press either the UP or DOWN arrow pushbutton keys and select **DST OFF**.




Press the CANCEL pushbutton key to exit. The Message Window will briefly display "Saving" and return to the "Ready" mode.

Date: March 9, 2007

Bulletin Number: SB# 07-05

Page: 3 of 3

PROCEDURE (750)

To enter the monitor's Setup menu, press and hold the LIMITS  and AUDIO/VISUAL  pushbutton keys while the monitor is being turned "ON" .

NOTE:

While in the Setup menu, if no pushbutton is pressed within 60 seconds, the monitor will automatically save all changes made and exit the Setup menu.

Press the DOWN arrow pushbutton key until the Time menu option is selected.

Press the AUDIO/VISUAL pushbutton key to move the cursor over to the Hour parameter in the right column.

Press either the UP or DOWN arrow pushbutton keys to change the Hour.

Press the LIMITS pushbutton key to move the cursor back to the previous column.

Press the DOWN arrow pushbutton key until the DST menu option is selected.

Press the AUDIO/VISUAL pushbutton key to move the cursor over to the DST options in the right column.

Press either the UP or DOWN arrow pushbutton keys and select OFF.

Press the RETURN pushbutton key to exit and return to the Main display.